

Reference

Direct digital X-ray with **Amadeo** in the joint veterinary practice of Dr. Beate Fischer-Wackes and Dr. Jörg Wackes in Gundelfingen, Germany



Veterinary group practice

Dr. Jörg Wackes
Dr. Beate Fischer-Wackes

79194 Gundelfingen, Germany
Tel. +49 761-5950015
info@wackes-tieraerzte.de
www.wackes-tieraerzte.de

 Amadeo Systems

Reference

Dr. Beate Fischer-Wackes and Dr. Jörg Wackes offer all the services of a practice for small animals, including X-ray imaging, ultrasound and endoscopy. The focus is on the fields of dermatology, gold implantation, (endoscopic) surgery, dentistry and physiotherapy. Several hundred X-ray images are taken each quarter.

Since early 2011, the practice has been working with a direct digital X-ray system. The Amadeo DR system includes a patient table with a floating tabletop, the X-ray generator which is integrated in the X-ray table to save space, and a DR Tech Flaatz 560 detector.

“Before we purchased the new X-ray system we made enquiries amongst our colleagues, a number of whom recommended solutions by OR Technology”, says Dr. Jörg Wackes. “Our previous X-ray system was conventional. Using direct digital X-ray imaging with a detector made our workflow significantly faster”, he explains the advantages of the new X-ray system. “The image quality in full image mode is very good. Image processing and image weighting are available and useful, allowing us to obtain more information from one image.”

The *dicomPACS®DX-R* acquisition software is operated via a touch screen. The generated images are then evaluated and archived using the *dicomPACS®vet* image management system.

Further information is available under
www.or-technology.com



Dr. Jörg Wackes,
Specialist veterinarian for
small animals and pets



“Diagnostic evaluation is simple and up-to-date and the large number of *dicomPACS®vet* functions, such as annotations, grid displays, enlargement, zoom, light/dark contrast changes and image export are most helpful. Thanks to the uncomplicated archiving option we have fast access to images even at a later stage. The integration with our practice software Vetinf is working smoothly”, says Dr. Wackes.

Service and consultation are also important issues for veterinarians. “We are very happy with the technical support offered by OR Technology”, comments Dr. Wackes. “Any reported faults are processed by friendly telephone staff. In urgent situations, the problem is forwarded immediately or dealt with promptly. All in all, we can count on clear directions and solution-orientated help from OR Technology.”



The Amadeo X-ray system of the veterinary group practice Dres. Wackes was installed in 2011 and no longer available. Below you will find our current model.



The space-saving full X-ray solution for your small animal practice – now "Made in Germany"!

Amadeo V mini II

The Amadeo V mini II X-ray unit has been specially designed for the requirements in veterinary medicine and is particularly suitable for small veterinary practices. The completely flat 4-way floating table without upstands will convince you. This veterinary X-ray system is a fully in-house creation "Made in Germany" by OR Technology and the result of close cooperation with veterinarians and university institutions.

The PC as well as a drawer for storage aids are inserted directly into the X-ray table to save space. The 32 kW high-frequency generator with rotating light visor is also integrated in the X-ray table. The foot brake on the table can be used to stop the movement of the table top and to operate the light on the collimator. A standard wall socket is sufficient to power the system. No major conversion measures are required.

The 180° swivel and height-adjustable touchscreen monitor allows convenient operation of the X-ray software (acquisition and reporting) in a very confined space. The integration of your practice management system is a matter of course for us.

A 3-stage LED status display on the X-ray stand provides simple visualisation of the operating status. With the help of the 2-step foot switch, the X-ray shot can be prepared and triggered.

Headquarters:

OR Technology (Oehm und Rehbein GmbH), 18057 Rostock, Germany, Neptunallee 7c
Tel. +49 381 36 600 500, Fax +49 381 36 600 555
www.or-technology.com, info@or-technology.com

OR Technology UK: Celtic SMR Ltd., Frederick House, Hayston View, Johnston
Haverfordwest, Pembrokeshire SA62 3AQ, United Kingdom
www.celticsmr.co.uk, sales@celticsmr.co.uk

Info-Hotline: +49 381 36 600 600



More information about Amadeo V-DR mini II you can be found here:
www.or-technology.com